

THERMOSKIN®

Thermoskin® Gloves are anatomically shaped to provide warmth and even compression for the temporary relief of arthritic pain. A textured inner layer provides additional grip for everyday activities and finger tips cut out, allow a better sense of touch.

- Promotion of increased blood flow facilitates enhanced recovery.
- Provides light but firm compression to counter act tissue swelling.
- · Clinically tested to increase the skin and muscle temperature.
- Provides temporary relief from pain/soreness associated with sports injuries, arthritis and RSI.
- · Increases elasticity and reactivity of the muscle.

TRIOXON® ADVANTAGE

Exclusive 3 dimensional lining that provides insulation and wicking of moisture via air circulation. This allows the skin to remain well oxygenated and comfortable for extended periods.



SIZE GUIDE Measure around the hand and knuckles



	XS	S	M	L	XL	XXL
СМ	15-17	18-20	21-23	24-26	27-29	30+
INS	6-634	7-734	8-83/4	914-10	104-1112	11114+

Position the product on the right and/or left hand so that the fastener strap is wrapped around the lower forearm. Secure the strap.

Do not wear within the first 72 hours following an acute injury. This product contains a synthetic fibre, which may cause allergic reactions. Should an irritation or rash develop, discontinue use and seek medical advice. If pain persists, discontinue use and consult your medical professional.





ммм. фытолими сот

Free Call: 1800 188 019 UNITED PACIFIC INDUSTRIES PTY LTD

.bneliedT ni 9beM

workmanship or materials. guarantee against faulty covered by a twelve month Your Thermoskin® product is Advena Ltd Tower Business Centre 2nd Floor, Tower Street Swatar, BKR 4013 Malta



bleach or dry in direct sunlight. washing bag. Machine wash in cold water. Drip dry. Do not wring, twist, tumble dry, Recommend placing product in a delicates Wash before wear.

Mid Layer: Rubber Foam, Inner Lining: Polyester: (Does not contain Latex). Outer Lining: Nylon,