español

SILOPAD™ Venda elástica Gel- Roll™ Gel- Roll™ Elastic Gel Bandage

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www.silipos.com

Made in U.S.A.
Licensed under U.S. Patent number
6,552,109

English

SILOPAD™ Gel- Roll™ Elastic Gel Bandage

- Product is an elastic bandage coated with Silipos' proprietary gel which contains a medical grade mineral oil (USP). The mineral oil gradually dissipates from the gel onto the skin to moisturize and protect skin from friction and pressure.
- Product conforms around bony prominences and may be cut without fraying.
- Product is dermatologist tested, hypo-allergenic, washable and reusable.

Indications:

- For edema control, cold therapy, and bursa problems.
- Helps to reduce muscle strains or arthritic pain and to control swelling in the elbow.
- Ideal for contusions and sprains.
- Product protects against friction and pressure.
- Product may be chilled for cold therapy.
- May also be used to moisturize and soften dry skin, and to help reduce scar tissue by improving the mobility and elasticity of the skin after scarring.

Directions for Use:

- Wash and dry affected area.
- Carefully wrap product around area with desired amount of compression.
- Product may be cut and placed on the affected area to provide moisturization and protection.
- Remove product at least 3 to 4 hours daily to allow skin to breathe.

Recommended Care:

We recommend that you hand wash product with mild soap and water. Rinse thoroughly. Then, air dry product and lightly dust gel with talcum powder after drying. Do not bleach product.

WARNING: Do not place product on an open wound. If irritation, discomfort or poor circulation occurs, discontinue use and notify your physician immediately. KEEP OUT OF REACH OF CHILDREN.

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